



MAGNOLIA
CHEMICAL PROFESSIONALS

**INFECTION PREVENTION
& CONTROL DENTAL
SUCTION SYSTEMS**

MAGNOLIA DISINFECTION
AND HYGIENE RANGE

WORKING AS ONE

At Cattani, we established *Magnolia Chemical Professionals*, an independent company fully committed to ensuring the optimal performance of Cattani equipment and protecting its users through a range of infection prevention and control solutions.

For over 50 years Cattani has been one of the leaders in the production of dental aspiration systems. The Magnolia range was designed specifically for disinfecting and cleaning dental suction systems.

It can be trusted to keep your practice hygienic for the safety of staff, patients and the protection of your dental suction system.



PULI-JET GENTLE 2.0

Non-Foaming Disinfectant for Dental Suction Systems

PULI-JET Gentle 2.0 non-foaming disinfectant for dental suction systems is specifically designed to disinfect and clean dental suction lines and dental suction systems. At Cattani, we understand the importance of convenience, hence we developed a solution that does not require alternating products or schedules making your daily infection prevention and control procedures for dental suction easy.

Details That Matter

- Disinfects and cleans
- Ease of use
- Phenol FREE
- Aldehyde FREE
- Lemon fresh scent



Easy, accurate and safe dosage chamber.



**1 ONE DISINFECTANT
ONE DILUTION
ONCE A DAY**



ENVIRONMENTAL PERFORMANCE

We continuously implement strategies to achieve long-term sustainability goals. PULI-JET Gentle 2.0's improved formula is part of our commitment to our environmental performance.



MINIMISE DEPOSITS, MAXIMISE PERFORMANCE

The anti-scaling agent helps minimise mineral deposits in the suction pipework and system, and helps keep your suction system operating at its best. Especially beneficial in hard water areas.



TIME TO ACT

Contact time is essential for the disinfectant to act on the biohazardous load. The included surfactant helps disperse disinfectant to more areas of the suction lines, amalgam separator and dental suction system for effective disinfection.



TAKE THE SHOCK OUT OF SUCTION SYSTEM MAINTENANCE

PULI-JET Gentle 2.0 in conjunction with the Pulse Cleaner provides effective cleaning and disinfection of dental suction lines and suction systems without the need to perform shock treatments.

HYGIENE SOLUTIONS FOR YOUR DENTAL PRACTICE

Magnolia has a comprehensive range of solutions to keep your suction system and suction lines operating at their best, while helping protect your most valuable assets – your people.




ANTIFOAMING DISINFECTANT TABLETS – CONTINUOUS DISINFECTION ALL DAY

Dual action, antifoaming function and slow release disinfection.

Reduces foam that can occur through dental procedures. A large amount of dense liquids such as blood, heavy saliva and mucus may cause foam in the suction system. Antifoaming and Disinfectant Tablets minimise this foam but also continuously disinfect the suction lines throughout the day.

Each pack contains 50 tablets, with one tablet required per dental chair per day.

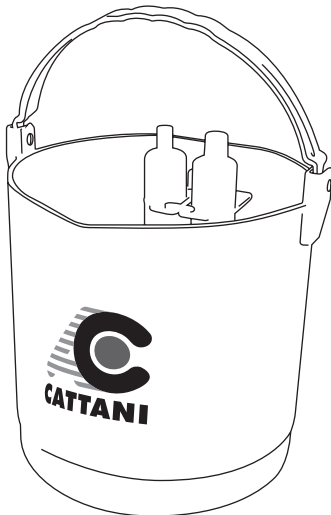

PULSE CLEANER – THE PERFECT PARTNER FOR DISINFECTING

The Pulse Cleaner automatically aspirates PULI-JET Gentle 2.0 disinfectant solution through the suction system by creating air and fluid turbulence, evenly and widely distributing the disinfectant for effective cleaning and disinfection of the suction lines, amalgam separators and dental suction systems.



LUBRI-JET SILICONE LUBRICANT – SUCTION SYSTEM HEALTH

LUBRI-JET is a silicone-based lubricant supplied in a handy dropper bottle, ideal for lubricating rubber O-rings and seals, which increases their lifespan and is part of the weekly preventative maintenance of your Cattani dental suction system.

Each pack of LUBRI-JET contains four 30ml bottles.



STEP-BY-STEP SUCTION SYSTEM MAINTENANCE

Point your smart phone camera at the QR code to view our short instructional videos.

BEST PRACTICE DENTAL SUCTION INFECTION PREVENTION & CONTROL

The Magnolia range provides dental clinics with a holistic solution for dental suction system infection prevention and control.



DISINFECTION DAY AND NIGHT

Working in harmony, the Antifoaming Disinfectant Tablets and PULI-JET Gentle 2.0 non-foaming dental disinfectant protect your patients, staff, end-users, and suction system by providing effective disinfection throughout the day with Antifoaming Disinfectant Tablets, and overnight with PULI-JET Gentle 2.0.

Antifoaming Disinfectant Tablets work independently from PULI-JET Gentle 2.0 and are an extra layer of protection for your patients, staff, and suction system. They provide continuous disinfection throughout the day with the added benefit of managing foam caused by certain dental procedures that produce a lot of blood, saliva and mucus.

When these two solutions are used as part of a day and night procedure, they provide a best practice solution for suction system disinfection. This helps protect the suction system and ensures optimal performance of the system over its lifetime.



Correct and regular disinfecting with Magnolia protects your suction system and helps ensure optimal performance.

Sales and Technical Support:



Australia
+61 3 9484 1120
cattani@cattani.com.au
www.cattani.com.au



New Zealand
0800 68 22 88
cattani@cattani.co.nz
www.cattani.co.nz

Company Address: 280 Dundas Street, Thornbury VIC 3071

